

RECOMMENDATION OF USE

1

- Although the Mobi-Chair floats, it is not a boat. The sitting position is stable and you won't slip out of the Mobi-Chair under normal conditions of use. For instance, for safety reason, we recommend to avoid using the wheelchair in adverse weather and other poor conditions.

2

- A life jacket is required for non-swimmers (this may also be required by law). The Mobi-Chair is not intended for areas with current or strong waves.

3

- The maximum limitation weight of the user is 150kg (approximately 330 pounds).

4

- Do not use part that can be detached, such as armrests, for support or any other reasons other than intended purposes.

5

- Avoid turning suddenly and never try to climb or descend a ramp diagonally

